



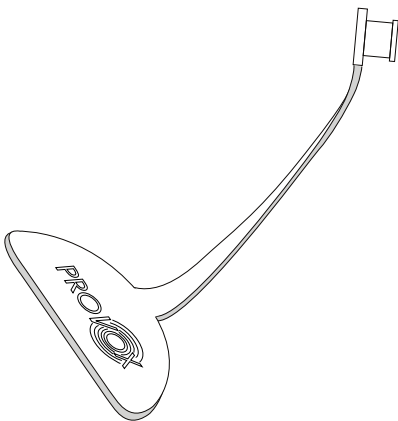
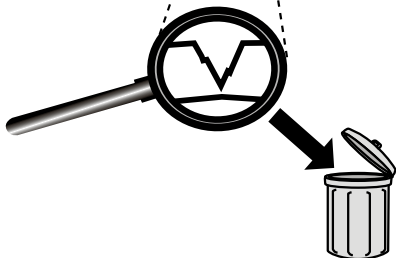
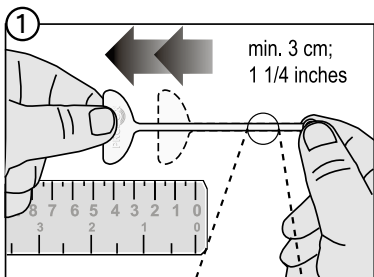
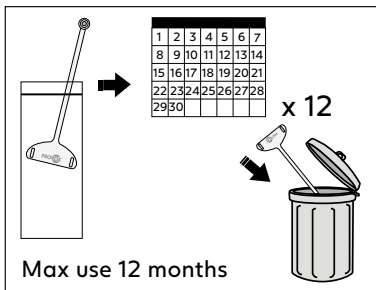
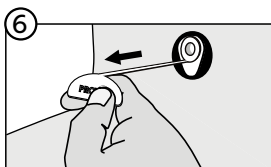
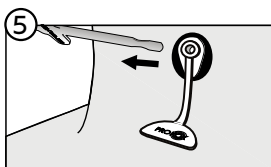
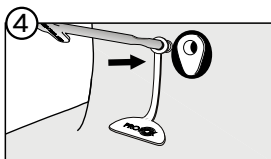
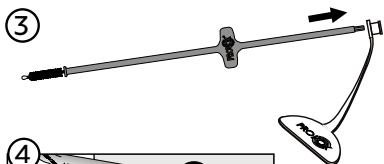
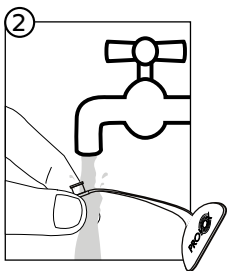
PROVOX[®] Plug

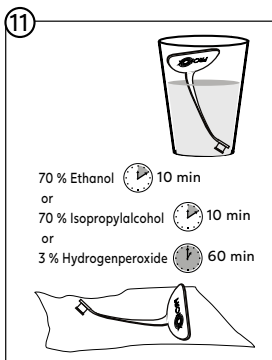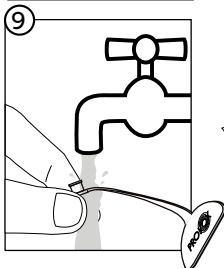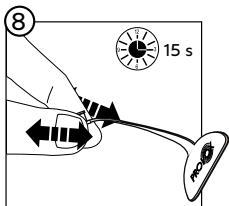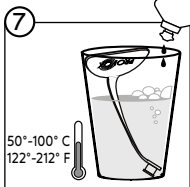2 / 250 ml (8,5 fl oz)

Disclaimer

Atos Medical offers no warranty - neither expressed nor implied - to the purchaser hereunder as to the lifetime of the product delivered, which may vary with individual use and biological conditions. Furthermore, Atos Medical offers no warranty of merchantability or fitness of the product for any particular purpose.

Patents and trademarks

Provox® is a registered trademark owned by Atos Medical AB, Sweden.

For information about protective rights (e.g. patents), please refer to our web page www.atosmedical.com/patents.

Manufacturer; Hersteller; Fabrikant; Fabricant;
 Produttore; Fabricante; Fabricante; Tillverkare; Pro-
 ducent; Produsent; Valmistaja; Framleiðandi; Tootja;
 Ražotājs; Gamintojas; Výrobce; Gyártó; Výrobca;
 Proizvajalec; Producent; Producător; Proizvođač;
 Proizvođač; Κατασκευαστής; Производител;
 Üretici; მწარმოებელი; Ἀρτοποιητής; İstehsalçı;
 Производитель; Produzen; Pengilang; 製造業者; 제
 조사; 製造商; 製造商; ייצור; الجهة المصنعة

Date of manufacture; Herstellungsdatum; Productieda-
 tum; Date de fabrication; Data di fabbricazione; Fecha
 de fabricación; Data de fabrico; Tillverkningsdatum;
 Fremstillingsdato; Produksjonsdato; Valmistuspäi
 vä; Framleiðsludagsetning; Valmistamise kuupäev;
 Ražošanas datums; Pagaminimo data; Datum výroby;
 Gyártás időpontja; Dátum výroby; Datum proizvodn-
 je; Data produkcji; Data fabricației; Datum proizvodn-
 je; Datum proizvodnje; Ημερομηνία κατασκευής;
 Дата на производство; Üretim tarihi; წარმოების
 თარიღი; Ἀρτοποιημάτων ἡμερομηνία; İstehsal tarixi;
 Дата изготовления; Tanggal produksi; Tarikh pem-
 buatan; 製造日; 제조일자; 製造日期; 製造日期;
 ייצור; تاريخ التصنيع

Use-by date; Verfallsdatum; Houdbaarheidsdatum;
 Date limite d'utilisation; Data di scadenza; Fecha de
 caducidad; Data de validade; Används senast; Anv-
 endes inden; Brukes innen; Viimeinen käyttöpäivä;
 Síðasti notkunardagur; Kölblik kuni; Derīguma

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1. Descriptive Information

1.1 Intended use

Provox Plug is intended to seal the inner lumen of Provox Voice Prostheses, Provox2 Voice Prostheses and Provox ActiValve and therefore stop leakage of both air and fluids through the voice prosthesis.

1.2 Description of the device

Provox Plug is a first-aid tool for temporarily stopping leakage through the voice prosthesis. The device is inserted into the opening of Provox voice prosthesis. The medallion end can be taped to the skin if desired.

1.3 Precaution

Provox Plug is intended for single patient use. Reuse in another patient may cause cross contamination.

2. Instructions for use

2.1 Operation instruction

Check device integrity prior to each use by pulling the strap see fig. 1.

For instructions on how to use the product, please see fig. 2-6.

2.2 Cleaning and disinfection

Clean Provox Plug after each use, or if it is changed during use according to Fig. 7-10.

Fig. 8, Rub the device between your fingers for 15 seconds.

DO NOT use any water other than drinking water to clean and rinse the device.

Disinfect Provox Plug at least once a day if it is removed (Fig. 11) with one of the following methods:

- Ethanol 70% for 10 minutes
- Isopropylalcohol 70% for 10 minutes
- Hydrogenperoxide 3% for 60 minutes

CAUTION: DO NOT clean or disinfect by any other method as this might cause product damage and injury. If Provox Plug looks dirty or has air dried in an area with a risk of contamination the device should be both cleaned and disinfected before use.

A risk of contamination could exist if the device has been dropped on the floor or if it has been in contact with a pet, someone with a respiratory infection, or any other gross contamination.

During hospitalization it is important to both clean and disinfect Provox Plug after use but also before use since there is an increased risk of infection and contamination. In a hospital it is better to use sterile water for cleaning and rinsing, rather than drinking water.

2.3 Disposal

Always follow medical practice and national requirements regarding biohazard when disposing of a used medical device.

2.4 Reporting

Please note that any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the national authority of the country in which the user and/or patient resides.

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