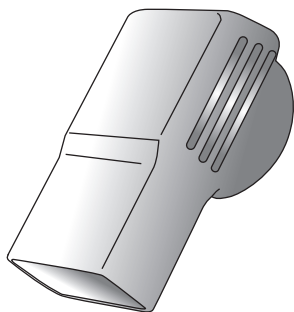
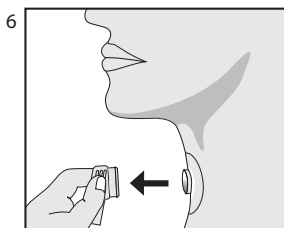
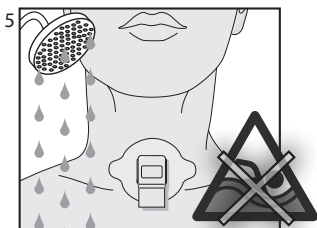
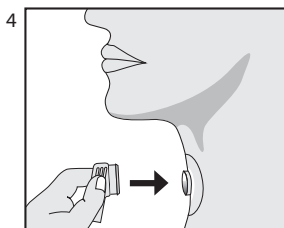
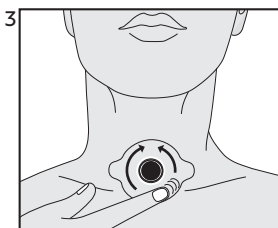
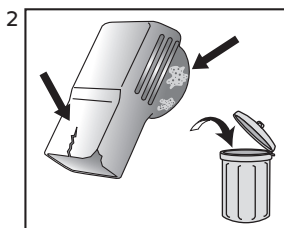
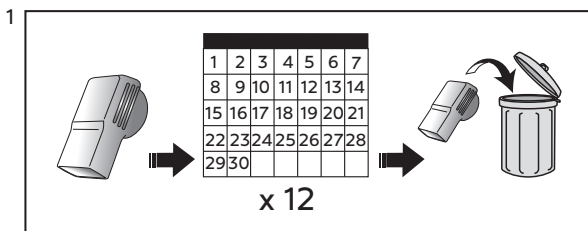
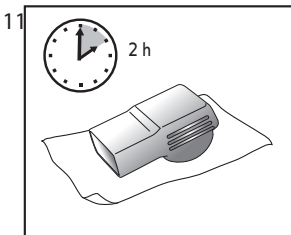
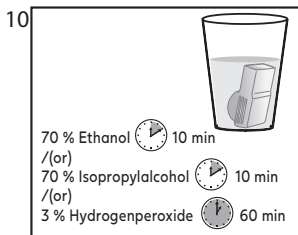
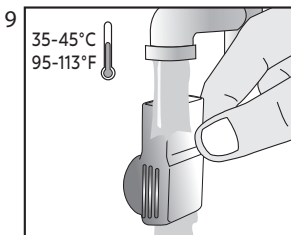
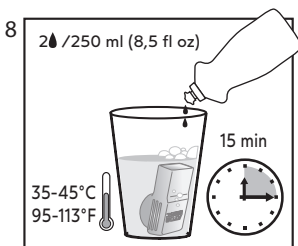
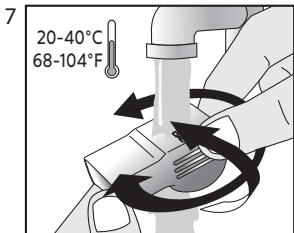
PROVOX[®] ShowerAid

Provox Adhesives

Disclaimer

Atos Medical offers no warranty – neither expressed nor implied – to the purchaser hereunder as to the lifetime of the product delivered, which may vary with individual use and biological conditions. Furthermore, Atos Medical offers no warranty of merchantability or fitness of the product for any particular purpose.

Compatible Products; Kompatible Produkte; Compatibele producten; Produits compatibles; Prodotti compatibili; Productos compatibles; Produtos compatíveis; Kompatibla produkter; Kompatible produkter; Kompatible produkter; Yhteensopivat tuotteet; Samhæfar vörur; Ûhilduvad tooted; Suderinami gaminiai; Kompatibilní produkty; Kompatibilis termékek; Kompatibilné produkty; Združljivi izdelki; Zgodne produkty; Produse compatibile; Kompatibilni proizvodi; Kompatibilni proizvodi; Συμβατά προϊόντα; Совместимые изделия; Uyumlu Ürünler; Համատեղելի արտադրանքներ; Produk yang Serasi; 互換性のある製品; 호환 가능한 제품; 相容產品; מוצרים תואמים; المنتجات المتوافقة

Provox StabiliBase
Provox XtraBase
Provox StabiliBase OptiDerm
Provox Flexiderm
Provox Optiderm
Provox LaryTube
Provox LaryButton

Contents

EN - ENGLISH	6
DE - DEUTSCH	7
NL - NEDERLANDS	8
FR - FRANÇAIS	10
IT - ITALIANO	11
ES - ESPAÑOL	13
PT - PORTUGUÊS	14
SV - SVENSKA	16
DA - DANSK	17
NO - NORSK	18
FI -SUOMI	20
IS - ÍSLENSKA	21
ET - EESTI	22
LV - LATVIESU	24
LT - LIETUVIŲ KALBA	25
CS - ČESKY	27
HU - MAGYAR	28
SL - SLOVENŠČINA	29
SK - SLOVENČINA	31
PL - POLSKI	32
RO - ROMÂNĂ	34
HR - HRVATSKI	35
SR - SRPSKI	37
EL - ΕΛΛΗΝΙΚΑ	38
TR - TÜRKÇE	39
HY - ՀԱՅԵՐԵՒՆԻՍԿ	41
RU - РУССКИЙ	42
MS - BAHASA MELAYU	44
JA - 日本語	46
KO - 한국어	47
ZHTW - 繁體中文	48
HE - עברית	51
AR - العربية	53

1. Intended use

Provox ShowerAid is used to temporarily replace the HME during showering. The ShowerAid can be placed in all Provox appliance holders.

2. Contraindications

The device should only be used in accordance with the Instructions for Use. Patients without the physical, cognitive, or mental ability required to attach, remove or operate the device themselves, should not use the device independently and should only use it if they are under sufficient supervision of a clinician or a trained caregiver.

3. Warnings

- Do not reuse the device between patients as it may cause cross-contamination. The device is intended for single patient use only.
- Do not use non-compatible or non-original devices as it may cause personal injury or damage the device.
- Do not dismantle or alter the device as it may cause personal injury or damage the device.
- Do not use a dirty or damaged device as it may cause infection (Fig. 2).
- Do not use the device when swimming/bathing as it may lead to aspiration of water which may result in aspiration pneumonia (Fig. 5).

4. Instructions for use

4.1 Operating instructions

Make sure that the adhesive is securely attached to the skin (Fig. 3). Attach Provox ShowerAid before showering (Fig. 4). To remove, hold the device and pull sideways until the device disconnects (Fig. 6).

4.2 Cleaning and disinfection

Clean and disinfect Provox ShowerAid before each use.
Only use drinking water to clean and rinse the device.

1. Rinse Provox ShowerAid in water (Fig 7).
2. Place Provox ShowerAid in water-based mild soap solution for 15 minutes (Fig 8).
3. Rinse Provox ShowerAid in water (Fig 9).
4. Disinfect Provox ShowerAid (Fig. 10) with one of the following methods:
Ethanol 70% for 10 minutes or Isopropylalcohol 70% for 10 minutes or Hydrogenperoxide 3% for 60 minutes.
5. Let Provox ShowerAid dry at least for 2 hours or until it looks dry (Fig. 11).

5. Storage instructions

Store Provox ShowerAid in a clean and dry container at room temperature. Protect from direct sunlight.

6. Device lifetime and disposal

Provox ShowerAid may be used for a maximum period of 12 months (Fig. 1). If the device shows any signs of damage, it shall be replaced earlier (Fig. 2). Always follow medical practice and national requirements regarding biohazard when disposing of a used medical device.

7. Reporting

Please note that any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the national authority of the country in which the user and/or patients resides.

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UK Responsible Person
Atos Medical UK Ltd Tottle Road
Cartwright House
Nottingham
Nottinghamshire NG2 1RT
England United Kingdom

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XXXXX, NN
YYYY-MM-DD

XXXXX, NN = Reference number, Version number;
YYYY-MM-DD = Date of issue;

THIS PAGE IS DELIBERATELY LEFT EMPTY

THIS PAGE IS DELIBERATELY LEFT EMPTY

THIS PAGE IS DELIBERATELY LEFT EMPTY

THIS PAGE IS DELIBERATELY LEFT EMPTY

90726, 08
2023-06-12

Atos
atosmedical.com

Atos Medical AB
Kraftgatan 8, SE-242 35 Hörby, Sweden
Tel: +46 (0)415 198 00 • info@atosmedical.com

© Coloplast A/S. All rights reserved.
Atos and the Atos Medical logo are trademarks of Coloplast A/S.